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EquiCrown compression bandages for horses are made of a very robust and durable knitted fabric. However, avoid any contact with pointed or sharpedged items to prevent damage to the bandages.

Applying the EquiCrown compression bandage with zipper

1. Place the bandage around the horse leg starting at the back of the leg. The fastener should point to the outer side. The fetlock needs to be placed into the provided bulge.
2. First close the velcro fastener to pre-set the bandage.
3. Reopen the top velcro fastener to fasten zipper, continue to zip the bandage downward while unfastening and then securing velcro over the closed zipper.
4. The end of the zipper should be folded back and secured under the velcro fastener.
5. If necessary adjust the bandage for it. The zipper needs to point to the outer side of the horse leg.

Applying the EquiCrown compression bandage with hook fastener

1. Place the bandage around the horse leg starting at the back of the leg. The fastener should point to the outer side. The fetlock needs to be placed into the provided bulge.
2. Starting from the top, fasten the hooks.
3. You may find it necessary to tighten the bandage by one row of hooks during the course of the treatment.

Receive the best tips on how to put on EquiCrown bandages in the easiest way.

equicrown.de/putting-on-compression-bandages

Please note

- Use EquiCrown compression bandages only for the intended indications and after consultation with your vet or animal therapist.
- The duration of wear of our bandages must be determined in consultation with the vet.
- Ensure that no contraindications to the use of compression exist.
- As therapy progresses or the condition changes, it may be necessary to remeasure/repurchase a new EquiCrown compression bandage.
- EquiCrown compression bandages are mostly resistant to oils, ointments, clay, skin moisture and other environmental influences.
- All products in the EquiCrown SkinCare range can be applied under the bandages.
- Wash the EquiCrown compression bandages regularly to maintain consistent compression.
- Store the bandages in a dry place at room temperature.
- Do not expose the bandage to extremely hot or cold temperatures.

- If the bandages get damaged, please consult your dealer. Do not repair the bandages yourself as this could impair the quality and effectiveness.

Care and washing instructions

- Before washing we recommend that all hooks and zippers are closed and turned inside out.
- The bandages should be washed separately the first time (the dye may run).
- Please do not use any softener!
- You can reduce the drying time by placing the washed bandage on a thick towel, rolling the towel up tightly, and firmly pressing out excess moisture. Do not leave the bandages in the towel.
- In case of daily use, EquiCrown compression bandages should be washed at least once weekly on a delicate cycle (40 °C).
- If washing by hand, rinse thoroughly.
- Do not wring! Line dry the bandages.
- Do not dry on a heater or in the sun.
- If tumble-dry, please use the delicate cycle.
- Do not dry clean.
- Please refer to the washing symbols on the label sewn into the bandages.

Material composition

Details are provided on the label sewn into the EquiCrown compression bandage.

Indications EquiCrown compression bandages

- preventative method during long standing periods (e.g. stall rest, transport)
- faster recovery
- swollen legs
- inactivity oedema
- supports chronic lymph drainage disorders
- soft windgalls
- (chronic) tendon injuries / disorders
- after fetlock injury
- post-operative to prevent from swellings
- post-operative and post-traumatic oedema
- post phlegmon
- scar treatment
- wound treatment in conjunction with wound dressing as substitute for medical wrap bandage (can be used in combination with EquiCrown Skin Care)

Indications for the EquiCrown compression bandage with silver thread

(with knitted-in antibacterial silver thread)

Same indications as for EquiCrown compression bandages, additionally:

- skin irritations (e.g. mud fever)
- optimizes the skin flora
- post injury treatment (always in conjunction with wound dressing)

Additional information for products containing silver

Since silver can conduct heat and electricity, direct contact with electrical sources should be avoided.

Absolute contraindications

- acute infections accompanied by fever (risk of bacterial spreading)
- cardiac insufficiency
- hyperthyroidism (excessive thyroid function)
- tumors that block lymphatic drainage

Relative contraindications

- venous obstruction edema (cardiac insufficiency must be excluded)
- acute phlegmon (risk of bacterial spreading)
- tumors outside the lymphatic system (risk of metastasization)

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

When sized and applied correctly, the risk of side effects with compression bandages is very low. Poorly seated bandages (e.g. after sizing errors) or incorrect application of the bandage, can lead to pressure points. Furthermore, particular horses may have a higher sensitivity to pressure and should only be treated with compression for a few hours at a time. If you notice pressure points on your horse, take off the bandage immediately. Please contact our customer service department to rule out possible application or sizing errors. If the bandages have been medically prescribed, please contact your veterinarian / animal therapist to discuss possible adjustments to the treatment. Compression can lead to dry or flaky skin. This can be treated using suitable skin care ointments. No intolerance to the materials used in our bandages has been observed to date. However, if negative alterations (e.g. skin irritations) should occur during use, please contact your veterinarian / animal therapist immediately. If an incompatibility should be known for your horse against one or several contents of this product, please consult with your veterinarian / animal therapist before use. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse of the bandage.

Disposal of bandages

Bandages can be disposed of in household waste. Please observe the applicable local regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany